

BIONOR™ MONOTAYL



A rapid
diagnostic test
for identification of
Taylorella equigenitalis

The difference
could be CEM




TEST PERFORMANCE



Place one drop of
test reagent in the
black circle of the
analysis card.

Look for
suspicious
colonies.

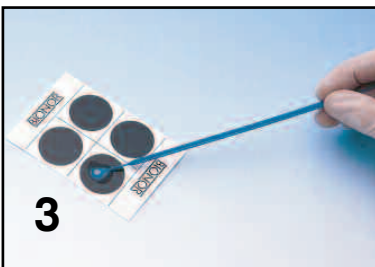

Transfer some of
the colonies to be
tested to the drop
of reagent and
mix with a loop
for 30 seconds.

Tilt the analysis
card 2-3 times
and look for
agglutination.
Positive reaction
(left).
Negative reaction
(right).



KIT CONTENTS AND ORDERING INFORMATION



Pack with reagents for the
performance of 50 tests.

Test reagent
Control reagent
Analysis cards
Instructions

Prod. code
DG 020

Description
MONOTAYL 50 test



Bionor Laboratories AS
Hollenderigt 3
NO-3732 Skien, Norway

Tel: +47 35 90 85 30
E-mail: bionor@bionor.com
Web: www.bionorlaboratories.com